WWW.MICROLIFE.COM  
E-MAIL: MSA@MICROLIFEUSA.COM  
FAX: +1 772 442 5377  
TEL: +1 772 442 5353  
CLEARWATER, FL 33755, USA

1617 GOLF TO BUY BLDG., 2ND FLOOR STE A  
MICROLIFE USA, INC.  
TEL: 1492 1288  
TAMPA, FL 33613, USA  
MICROLIFE CORPORATION

WWW.MICROLIFE.COM  
E-MAIL: ADMIN@MICROLIFE.CH  
FAX: +86 2 8797 1283  
TEL: +86 2 8797 1288  
TAMPA, FL 33613, USA  
MICROLIFE (CHINA) S.R.L.  
ESPRESSO STRASSE 139  
MICROLIFE AG  
DISTRIBUTOR

WWW.MICROLIFE.COM  
E-MAIL: ADMIN@MICROLIFE.CH  
FAX: +86 2 8797 1283  
TEL: +86 2 8797 1288  
TAMPA, FL 33613, USA  
MICROLIFE (CHINA) S.R.L.  
ESPRESSO STRASSE 139  
MICROLIFE AG  
DISTRIBUTOR

WWW.MICROLIFE.COM  
E-MAIL: ADMIN@MICROLIFE.CH  
FAX: +86 2 8797 1283  
TEL: +86 2 8797 1288  
TAMPA, FL 33613, USA  
MICROLIFE (CHINA) S.R.L.  
ESPRESSO STRASSE 139  
MICROLIFE AG  
DISTRIBUTOR

WWW.MICROLIFE.COM  
E-MAIL: ADMIN@MICROLIFE.CH  
FAX: +86 2 8797 1283  
TEL: +86 2 8797 1288  
TAMPA, FL 33613, USA  
MICROLIFE (CHINA) S.R.L.  
ESPRESSO STRASSE 139  
MICROLIFE AG  
DISTRIBUTOR

Medical compressor nebuliser Model NEB 10

- 1 Medical compressor nebuliser
2 Power lead
3 ON/OFF Switch
4 Air filter compartment
5 Nebuliser
6 Air hose
7 Mouthpiece for inhalation through the mouth
8 Adult face mask
9 Child face mask
10 Nose piece for inhalation through the nose
11 Replacing air filter
12 Assembling nebuliser kit
13 Operating diagram

Dear Customer,
This professional nebuliser is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract.
The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.
Please read through these instructions carefully so that you understand all functions and safety information.

- Read the instructions carefully before using this device.
Type B applied part
Conformity mark
Class II equipment

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions.
This device is not suitable for anaesthesia and lung ventilation.
This device should only be used with original accessories as shown in these instructions.
Do not use this device if you think it is damaged or notice anything unusual.
Never open this device.
This device comprises sensitive components and must be treated with caution.

- Ensure that children do not use this device unsupervised: some parts are small enough to be swallowed.
Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
Use of this device is not intended as a substitute for a consultation with your doctor.

2. Contraindications

For type, dose, contraindication and regime of medication follow the instructions of your doctor or respiratory therapist. Please carefully read the instructions of the drug before using it.

3. Preparation and Usage of this Device

- Assemble the nebuliser kit. Ensure that all parts are complete. Use the wheel on the upper part of the nebuliser to select the position most suitable for the therapy to be performed.
Fill the nebuliser with the inhalation solution as per your doctor's instructions.
Connect the nebuliser with the air hose to the compressor and plug the power lead into the socket.

4. Choosing between 3 different Nebulisation Rates

The nebuliser generates an aerosol with variable characteristics to adapt to the patient's respiratory pathology. It can produce an aerosol with the granulometric characteristics that will deposit the maximum amount of the drug in the respiratory tract being treated.

- Upper air passages (nose, throat, etc.): Position the wheel towards the bottom at number 3.
Tracheal / bronchial tract: Position the wheel in the center at number 2.
Deep air passages: Position the wheel towards the top at number 1.

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3A Health Care S.r.l.